

RESCUE BREATHER

01RB1000

o-two controlled[™] ventilation

Mouth to Mouth RESUSCITATION AID (DISPOSABLE NOT FOR REUSE)

LOT 2023-07
PAT. N° 97/7008



EU Representative
Houben/Marcel
32 Rue Vin ve
4030 Li ge/Luik
Belgique

 **O-TWO MEDICAL TECHNOLOGIES INC.**

45A Armthorpe Road, Brampton, Ontario, Canada L6T 5M4
Tel. (905) 792-6896 Fax (905) 799-1339
www.otwo.com

This resuscitator is designed to offer protection to both the first aider and the patient when performing Mouth to Mouth Resuscitation.

INSTRUCTIONS FOR USE

- 1 Ensure it is safe to approach the casualty.
- 2 If no breathing sounds or movement are detected, place casualty on their back.

- 3 Tilt head back and extend jaw forward.

See figure A

- 4 Clear airway of any obstruction and keep it clear.

- 5 Loosen clothing around the neck



- 6 Unfold the plastic sheet that surrounds the device and place the oval side of the mouth piece of the resuscitator in the casualty's mouth.

See figure B



- 7 With finger and thumb, pinch shut the nostrils through the sheet while supporting the jaw with the other hand.

See figure C



- 8 Make a good seal over the device and against the face. Commence mouth to mouth resuscitation as required. See figure D. Ensure that the mouth and airway are kept clear of vomit and or secretions.



WARNING Do not overinflate, take special care with children and babies



- 9 Continue the resuscitation until medical assistance arrives or until breathing recommences when casualty should be placed carefully in the recovery position.

- 10 Remove immediately after use and avoid any direct contact with the casualty's side of the device. Discard safely using original packaging.

NOTE In case of possible spinal/head injury, when opening the airway, use modified jaw thrust method without head tilt.

NOTE These instructions are a guide only and no substitute for proper training.