



# O-Two Statvac® II Oxygen-Powered Aspirator

Oxygen-powered suction on the move

# O-Two Statvac® II

When there is a risk of airway compromise from fluids or foreign material in the airway, the O-Two Statvac® II oxygen powered aspirator will effectively remove the materials and reduce the risk of aspiration.

The O-Two Statvac II oxygen powered aspirator is capable of effectively suctioning excessive quantities of fluids or particulates from the patient's oro/naso-pharynx. This lightweight aspirator is designed to be attached to any 50 PSI DISS outlet (other connections available upon request).

The unique ON/OFF shuttle provides for fast actuation of the aspirator in an emergency.

Continuous suction is provided by depressing the green shuttle control. The O-Two Statvac® II aspirator can be quickly and easily turned off by depressing the red shuttle control button on the opposite side of the aspirator. The suction catheter can be quickly cleared of foreign material by covering the aspirator exhaust port with one finger, reversing the flow.

## Instructions for Use

1. Connect one end of the gas supply hose to the DISS inlet on the aspirator and the other end to a 50 PSI oxygen supply.
2. Turn the oxygen supply on.
3. Connect the suction tubing to the barbed nipple on the aspirator and ensure that the lid is secured to the collection jar.
4. Depress the green shuttle button to initiate suction flow and occlude the end of the clear plastic suction tubing to ensure that a vacuum is created.
5. Ensure that the collection jar remains upright to prevent fluids from blocking the ports in the lid during operation.
6. In the event that it becomes plugged by foreign material, the suction catheter can be cleared by covering the aspirator discharge port on the lid. This will create a reverse flow through the suction catheter.

**Do not suction for more than five seconds at a time as suctioning may deplete the patient's oxygen levels.**

7. Depress the red shuttle button immediately after each suctioning procedure to conserve the oxygen supply.
8. Repeat the suctioning as required, emptying the collection jar as necessary.

**Note:** In an emergency situation, excess fluid from the over-filled jar will automatically exhaust through the lid exhaust. This ensures no interruption in the suctioning procedure.

## Specifications

Supply Pressure:	50 PSI (3.4 BAR)
Vacuum Pressure:	300mm Hg +/- 10%
Vacuum Free Airflow:	30 L/min +/- 15%
Maximum Gas Consumption:	20 L/min
Operating Temperature Range:	-18 to 50°C (0 to 122°F)
Storage Temperature Range:	-40 to 60°C (-40 to 140°F)
Relative Humidity for Storage and Operating Use:	15 to 95%
Weight:	5.6 oz (160 gms)
Dimensions:	Dia. 3.0 in. (76mm) X H: 7.0 in (178mm)

## Features

- Provides a fast and effective method of clearing fluid and foreign material from the patient's mouth in an emergency
- Unique, positive, color coded ON/OFF action shuttle valve control for ease of operation
- Durable anodized aluminum body on a high-impact resistant 8oz (optional 12 oz) collection jar

## Cleaning and Disinfection Procedures

1. After use, rinse out contents of aspirator jar with water. Reconnect the jar to the aspirator lid, connect to an oxygen supply and suction clean, soapy water through the aspirator to clear all foreign material from the aspirator. Allow the clean water to flow through the discharge port.
2. Turn off the gas supply and remove the hose assembly from the aspirator lid. Dispose of the gasket and suction tubing in accordance with local protocols and immerse the collection jar and lid in soap and water and wash thoroughly.
3. Remove from the soapy water and rinse the components in clear water prior to immersing in a commercially available, legally marketed, cold liquid sterilant following the manufacturer's instructions.
4. Remove from the solution and rinse thoroughly.
5. Inspect for damage and replace parts as necessary.
6. Dry and reassemble the jar and lid replacing the gasket with a new part.
7. Test the aspirator for function by attaching to a regulated gas supply, depressing the green shuttle button and checking for vacuum. If no vacuum is generated, ensure that the gas supply is turned on, the collection jar gasket is present and the jar is fully tightened against the lid. Check all tubing and hoses for tight connections.
8. If function is not restored return, the aspirator to an O-Two authorized service center for service.
9. Replace the suction tubing and catheter with new parts as required.

## Ordering Information

### 01AS9500 Statvac II Oxygen Powered Aspirator

Incl., Six-foot hose, 8 oz collection jar, Yankauer suction tip and tubing

### 01AS9502 Statvac II Oxygen Powered Aspirator

Incl., Six-foot hose, 12 oz collection jar, Yankauer suction tip and tubing

### 17MP3051-cs Statvac II

Replacement 8 oz collection jar (Case of 10)

### 17MP3065-cs Statvac II

Replacement 12 oz collection jar (Case of 10)

### 17MP3052-cs Statvac II

Replacement lid gasket (Case of 10)



45A Armthorpe Road, Brampton, ON, Canada, L6T 5M4

Telephone: +1 905 792-OTWO (6896)

N.A. Toll Free: +1 800 387 3405 Facsimile: +1 905 799 1339

Web: [otwo.com](http://otwo.com) Email: [resuscitation@otwo.com](mailto:resuscitation@otwo.com)

Statvac is a registered trademark of O-Two Medical Technologies Inc.

15PL7017 Rev. 10\_10\_16