

RESCUE BREATHER® CPR POCKET VENTILATOR

RE-ORDER: #01PVI503W



Rev5 Dec 16, 2019



LOT



CONTENTS

1. Foldable seamless cushion mask with oxygen inlet
2. One way valve with filter
3. Elastic head strap


⚠ CAUTION ⚠

This product should be used by trained personnel.



O-TWO MEDICAL TECHNOLOGIES INC.

45A Armthorpe Road, Brampton
Ontario, Canada L6T 5M4
Tel. (905) 792-6896 Fax (905) 799-1339

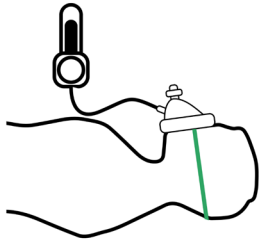


Connect the oxygen supply tubing when supplemental oxygen is required.

⚠ CAUTION ⚠


In the presence of oxygen, keep away from the presence of open flames, oil and grease

OPERATION FUNCTION

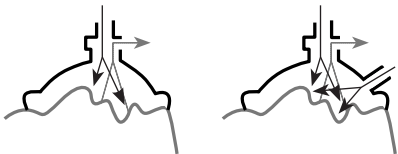


Inhalation/ Expiration
resistance less than
3.0 cm H₂O at 50 lpm

1

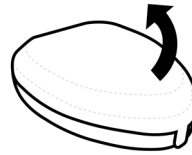

Open the case.

2

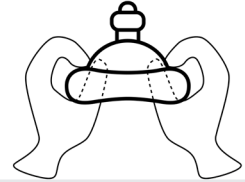


Push the mask from the inside to expand it.

3




Press the oxygen tubing plug before application of the mask to ensure seal and prevent any leakage.

4

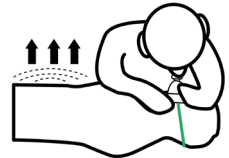

Give rescue breaths through the valve.

5

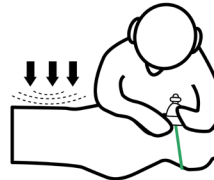

Continue CPR procedure accordingly.

Infants



Rotate the mask when using it in infants.

www.otwo.com

o_two™