

CAUTION

1. Federal law restricts this device to sale by or on the order of a physician.
2. Patient should be constantly monitored whenever this device is in use.
3. Store in clean dry conditions. Away from heat and light.
4. The device is designed for single use only. Reuse of this device might cause malfunction or contamination.

DIRECTION FOR USE

1. Choose the correct size (the correct size for a patient is determined by the distance between the corner of the mouth and the tragus).
2. Lubricate then insert with the curvature initially facing upward and then rotate as the the device is inserted halfway through so as to follow the curvature of the tongue.
3. In edentulous patients, it may also be inserted right from the outset with its curvature along the curvature of the tongue.
4. Once inserted, it is very important to support the jaw to maintain patency of the airway.

REF 01AM3000

R_x Only



LOT



O-TWO MEDICAL TECHNOLOGIES INC.

45A Armthorpe Road, Brampton, Ontario, Canada L6T 5M4
Tel. (905) 792-6896 Fax (905) 799-1339

www.otwo.com