


EASY GRIP®

Disposable Bag-Valve-Mask Resuscitator

“UNBELIEVABLE QUALITY AT AN UNBELIEVABLE PRICE”!

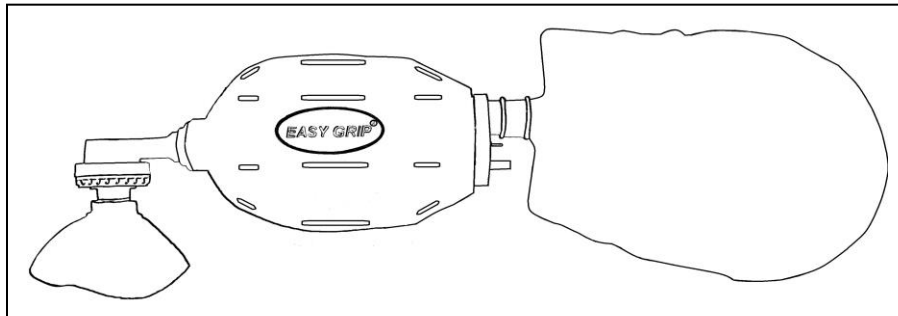

The EASY GRIP® Disposable Bag Valve Mask Resuscitator is a single use device for manual ventilation. It is designed for use in various clinical settings to provide respiratory support to the respiratory distressed or non-breathing patient.

The Child and Infant resuscitators come standard with a Pressure Relief Valve set at 40 cm H₂O which can be overridden by the operator.

When used with a supplemental oxygen supply, the device may provide 100% oxygen on every squeeze of the bag.

I. Pre-use function check

1. Inspect the EASY GRIP® resuscitator to ensure that all components are present and properly assembled.
2. Test for leaks by occluding the patient port completely squeezing the bag (Any leaks in the system may prevent the delivery of a sufficient volume to the patient).
3. Squeeze and release the EASY GRIP® hard a few times to ensure that air is moving through the valve system to the mask.
4. If using supplemental oxygen, attach the reservoir system to the bag refill port and ensure that the oxygen tubing is attached to an oxygen source with a flow rate of at least 15 L/min. Ensure that the collapsible reservoir system is fully extended to allow maximum oxygen storage.

II. Directions for use:

1. Select the appropriate EASY GRIP® resuscitator model for the size of patient to be ventilated.
2. Ensure that the patient's airway is clear of any obstructions and remains open by properly positioning the patient's head in accordance with local protocols.

3. Maintain a proper mask-to-face seal with one hand by lifting the chin upward with the last three fingers of the hand. Keep the index finger and thumb on top of the mask to form a tight seal around the patient's mouth and nose.
4. Gently squeeze the EASY GRIP® with the other hand until the chest rises, then release. Ventilate the patient with a steady squeeze and release of the EASY GRIP® sufficient time between ventilations to allow for full emptying of the patient's lungs.
5. If the child or infant EASY GRIP® is being used and the Pressure Relief override is required to be applied, depress the Pressure Relief Button and rotate 90° to lock in place. To unlock simply rotate the button until the arrow lines up with the arrow on the patient valve and release.
6. Safely dispose of the EASY GRIP® after use in accordance with local protocols.

Warnings:

The EASY GRIP® Disposable Bag Valve Mask Resuscitator should only be used by personnel trained in Cardio Pulmonary Resuscitation and in the use of this device.

Use only as directed. Improper use or unauthorized modification of this product may result in user or patient injury.

Caution:

Federal (USA) law restricts this device to sale by or on the order of a physician.



Oxygen concentration (%) delivered by Adult EASY GRIP® Resuscitator (values in parentheses are without an oxygen reservoir)						
O ₂ Input l/min	Tidal Volume (ml) x Ventilation Frequency (BPM)					
	600 X 12	600 x 20	750 X 12	750 X 20	1000 X 12	1000 X 20
5	82 (32)	58 (34)	65 (34)	50 (30)	55 (31)	45 (31)
10	97 (37)	80 (38)	97 (37)	97 (36)	88 (36)	62 (36)
15	99 (46)	97 (45)	99 (46)	99 (44)	97 (44)	90 (46)

Oxygen concentration (%) delivered by Child EASY GRIP® Resuscitator (values in parentheses are without an oxygen reservoir)			
O ₂ Input l/min	Tidal Volume (ml) x Ventilation Frequency (BPM)		
	70 X 30	200 X 30	300 X 30
5	96 (66)	59 (38)	45 (33)
10	97 (82)	97 (48)	69 (38)
15	97 (89)	97 (48)	97 (48)

Oxygen concentration (%) delivered by Infant EASY GRIP® Resuscitator (values in parentheses are without an oxygen reservoir)				
O ₂ Input l/min	Tidal Volume (ml) x Ventilation Frequency (BPM)			
	20 X 30	20 X 60	40 X 60	70 X 60
5	97 (75)	97 (72)	97 (59)	85 (52)
10	97 (75)	97 (78)	97 (78)	85 (61)
15	97 (95)	97 (92)	97 (82)	97 (73)

SPECIFICATION:

Storage Temperature Range: -40°C to 60°C (-40°F to 140°F) Inspiratory Resistance: 3.3cm H₂O
 Operating Temperature: -18°C to 50°C (0°F to 122°F) Expiratory Resistance: 2.2cm H₂O
 Patient Valve Dead space: 7.0 ml

	Adult	Child	Infant
Bag Volume:	1700 ml	470 ml	300 ml
Stroke Volume:	900 ml	250 ml	130 ml
Maximum Cycle Rate:	45 BPM	100 BPM	95 BPM
Pressure Relief:		40 cmH ₂ O	40 cmH ₂ O
Reservoir Volume:	2500 ml	2500 ml	1000 ml

ORDERING INFORMATION

01BM3500-Cs EASY GRIP® Disposable (Adult) c/w Universal Mask, Reservoir System and O₂ supply tubing (Cs/12)
 01BM3501-Cs EASY GRIP® Disposable (Adult) c/w Cuff Mask, Reservoir System and O₂ supply tubing (Cs/12)
 01BM3510-Cs EASY GRIP® Disposable (Child) c/w Universal Mask, Reservoir System and O₂ supply tubing (Cs/12)
 01BM3511-Cs EASY GRIP® Disposable (Child) c/w Cuff Mask, Reservoir System and O₂ supply tubing (Cs/12)
 01BM3520-Cs EASY GRIP® Disposable (Infant) c/w Cuff Mask, Reservoir System and O₂ supply tubing (Cs/12)



EU Representative:

Marcel Houben
 Rue Vinàve, 32
 4030 LIEGE Belgique



7575 Kimbel Street, Mississauga, ON, CAN, L5S 1C8
 Telephone 905 677 9410 N.A. Toll Free 800 387 3405
 Facsimile 905 677 2035 Web otwo.com
 E-mail: resuscitation@otwo.com