

# BREATHING CIRCUIT FOR EQUINOX® SERIES ANALGESIC GAS MIXING AND DELIVERY SYSTEMS

## INSTRUCTIONS FOR USE


1. Inspect packaging and circuit for visible signs of damage. Do not use if damaged.
2. Connect White Hose (Inspiratory Tubing) to device Output Connector.
3. Connect Pink Hose (Expiratory Tubing) to Scavenger System (if equipped).
4. Make sure connections are compatible and fit firmly.
5. Extend corrugated hoses to desired length.
6. Supplied Mouthpiece can be replaced with standard face mask. One size (Child-Adult) O-Two Universal Face Mask (02FM4999) is recommended.

## CAUTION

1. Federal Law in the U.S.A. restricts this device to sale by or on the order of a licensed practitioner.
2. This Single-use Circuit can be used with Single or Dual Circuit Analgesic Gas Mixing and Delivery Devices.
3. The use of a filter with the Disposable Circuit may increase inspiratory and expiratory resistance along with flow changes.
4. This circuit is designed for a single patient use only and cannot be disassembled. Do not attempt to clean, disinfect or re-use.  
Reusing this patient circuit may pose a risk of cross contamination while cleaning or disinfecting it may result in device malfunction.
5. Keep package sealed until use and store in clean dry conditions.
6. Dispose of patient circuit in accordance with local protocols.

## SPECIFICATIONS:

Intended patient group:	Adult	Overall length/size:	95 cm/3 feet (collapsed); 300 cm/10 feet (extended); ID: 22 mm
Rated flow:	180 Lpm p<0.2 Kpa	Working pressure:	-5 cmH <sub>2</sub> O to 0
Resistance to flow	0.01 cm H <sub>2</sub> O/ l/min (R@30 l/min)	Compliance:	1.28 ml/cmH <sub>2</sub> O (C@60 cmH <sub>2</sub> O)

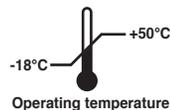

Rev. 5 - Feb. 2020



**O-TWO MEDICAL TECHNOLOGIES INC.**

45A Armthorpe Road, Brampton, Ontario, Canada L6T 5M4  
Tel. (905) 792-6896 Fax (905) 799-1339

[www.otwo.com](http://www.otwo.com)



MedNet EC-REP GmbH  
Borkstrasse, 10  
48163 Münster, Germany