

O-two Single Use CPAP



o_two controlled
ventilation™

O-TWO SINGLE-USE OPEN CIRCUIT CPAP DELIVERY SYSTEM

AWARD WINNING, COMPACT, "LOW FLOW", NON-INVASIVE VENTILATION FOR THE RESPIRATORY DISTRESSED PATIENT

CPAP has been proven to provide an effective method of treating respiratory distress from CHF. The use of CPAP is now being extended to other respiratory problems to provide symptom relief and avoid the need for intubation and ventilation of the patient and ICU admission. The O-Two Single-Use "Open CPAP" Delivery System provides incredibly accurate CPAP delivery for such a compact device. By minimizing the pressure drop on inspiration and the peak pressure on expiration, the O-Two Single-Use "Open CPAP" Delivery System produces a more uniform CPAP pressure throughout the respiratory cycle. This provides a lower work of breathing for the patient when compared to other, commonly used, pre-hospital devices.

A Luer port is located on the device to attach a pressure gauge. However, the accuracy of the CPAP level, at the fixed flow settings noted on the device, makes the use of a gauge unnecessary. The ambient air intake port and the location of the in-line oxygen hose are designed to eliminate the possibility of accidental occlusion, removing the risk of barotrauma due to the patient or rescuer inadvertently blocking the port. Ideal for the treatment of the respiratory impaired patient suffering from a range of obstructive pulmonary diseases, the O-Two Single-Use "Open CPAP" Delivery System is a simple, cost effective alternative to invasive ventilation and possible intensive care admissions.



SMALL AND LIGHTWEIGHT
Complete system weighs only 4oz (0.1 Kg)

LOW OXYGEN CONSUMPTION
Only 8 - 25L/min for 5 - 25 cm H₂O of CPAP

SIMPLE TO USE
Flow control adjustment to set CPAP level

WIDE CPAP RANGE
5 - 25 CM H₂O

DECREASED WORK OF BREATHING
Open circuit imposes no work on the patient

COMFORTABLE AND SECURE HEAD HARNESS
Soft, conforming Neoprene material does not slip.

COST EFFECTIVE
Low purchase price and low oxygen consumption compared to other devices.

EASY TO USE

The adjustment of the CPAP level is achieved by adjusting the output flow from your oxygen therapy regulator or wall outlet. The setting selections noted on the device provide an accurate constant airway pressure at each flow setting.

As this is an "Open" system, the device allows unrestricted inspiratory flows, since the patient has access to ambient air. Nebulizer treatments can be provided "in line", with the nebulizer positioned between the face mask and the CPAP unit.

FLOW RATE (L/min)	8	10	12	15	20	25
PRESSURE (cmH ₂ O)	5.0	8.0	10.0	15.0	20.0	25.0
OXYGEN (%)*	54	59	62	67	73	77
CYLINDER DURATION D Tank (415L) MIN	52	42	35	28	21	17

* Over a tidal volume range of 100 to 750ml

ORDERING INFORMATION



CONFIGURATION 1

CPAP system only

01CV0211-cs - o_two CPAP System (Large Adult)
c/w face mask (size 5) and head harness (Case/10)

01CV0212-cs - o_two CPAP System (Adult)
c/w face mask (size 4) and head harness (Case/10)

01CV0213-cs - o_two CPAP System (Child)
c/w face mask (size 3) and head harness (Case/10)



CONFIGURATION 2

CPAP system with Manometer

01CV0217-cs - o_two CPAP System (Large Adult)
c/w Adult Cuffed Facemask (size 5), Head Harness System, Flow Vector Valve with Manometer and Oxygen Supply Tubing (Case/10)

01CV0218-cs - o_two CPAP System (Adult)
c/w Adult Cuffed Facemask (size 4), Head Harness System, Flow Vector Valve with Manometer and Oxygen Supply Tubing (Case/10)

01CV0219-cs - o_two CPAP System (Child)
c/w Child Cuffed Facemask (size 3), Head Harness System, Flow Vector Valve with Manometer and Oxygen Supply Tubing (Case/10)



CONFIGURATION 3

CPAP system with Manometer and nebulizer

01CV0224-cs - o_two CPAP System (Large Adult) with Nebulizer
c/w Adult Cuffed Facemask (size 5), Head Harness System, pressure manometer, nebulizer with tubing and preset flow restrictor and "T" piece(Case/10)

01CV0225-cs - o_two CPAP System (Adult) with Nebulizer
c/w Adult Cuffed Facemask (size 4), Head Harness System, pressure manometer, nebulizer with tubing and preset flow restrictor and "T" piece(Case/10)

01CV0226-cs - o_two CPAP System (Child) with Nebulizer
c/w Adult Cuffed Facemask (size 3), Head Harness System, pressure manometer, nebulizer with tubing and preset flow restrictor and "T" piece(Case/10)

REPLACEMENT ITEMS



01CV0210-CS - O-Two Disposable Open Circuit CPAP Valve c/w oxygen supply tubing (Case/50)



01CV0220-CS - O-Two Single-Use CPAP Nebulizer System c/w 6 l/min Flow Adapter, T-Piece, Updraft Nebulizer with Oxygen Supply Tubing (Case/10)



02RT1301-CS - O-Two Single-Use CPAP Nebulizer T-Piece (Case/10)



01CV8009-CS - O-Two Replacement Neoprene Head Harness System (Case/5)



01CV8066-CS - O-two Manometer (Case/25)

SPECIFICATIONS

CPAP RANGE	5 - 25 cm H ₂ O
REQUIRED THERAPY FLOW RATE	8 - 25 L/min
DIMENSIONS (CPAP DEVICE ONLY)	1.6" dia x 2.3" (40.6mm dia x 58.3 mm)
WEIGHT WITH FACE MASK AND TUBING	4 Oz. (0.1 Kg.)
INPUT CONNECTION	Oxygen Therapy Barb
PATIENT FACE MASK CONNECTION	22ml
OPERATING TEMPERATURE	-18°C to +50°C (0°F to 122°F)
RELATIVE HUMIDITY FOR STORAGE AND OPERATING USE	-40°C to +60°C (-40°F to 140°F)

o_two controlled™
ventilation

www.otwo.com

45A Armthorpe Road, Brampton, ON, Canada, L6T 5M4

Telephone: +1 905 792-OTWO (6896) N.A. Toll Free: +1 800 387 3405

Facsimile: +1 905 799 1339 Email: resuscitation@otwo.com

CE 0120