


## 01BM1500 PEDI Ventilation Timing Light for Manual Resuscitators

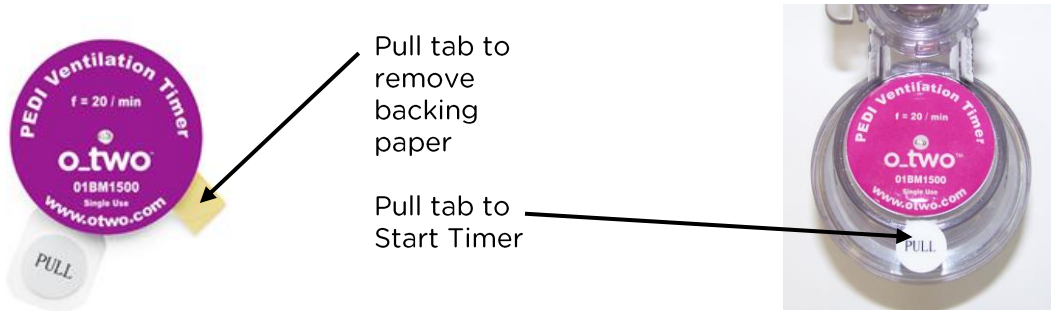

### DIRECTIONS FOR USE

- The PEDI Ventilation Timing Light comes with an adhesive backing with peel off cover paper to allow the attachment of the timing light to manual resuscitators.
- The Ventilation Timing Light is activated by simply pulling the tab as indicated.
- Follow the timing light by initiating the ventilation when the light turns “ON” and slowly delivering the breath over the duration of the “ON” time.
- When the light goes “OFF”, release the bag squeeze allowing the bag to refill and the patient to exhale.

### CAUTION

- Do not attempt to deliver the breath between compressions but provide the breath over the course of 2 - 3 compressions.
- Store in clean dry conditions.

### SPECIFICATIONS

“ON” time:	1.0 seconds
“OFF” Time:	2.0 seconds
Ventilation Rate:	20 breaths per minute
Battery type:	Li/MnO <sub>2</sub> (Lithium Manganese Dioxide) Coin Battery
Battery Life:	3 years
Storage Temperature:	-20 to 50 °C
Operating Temperature:	- 5 to 40 °C

### ORDERING INFORMATION

01BM1500-CS PEDI Ventilation Timing Light - Individually packaged (c/s 50)



**O-Two Medical Technologies Inc.**

45A, Armthorpe Road, Brampton, Ontario, Canada L6T 5M4

Toll Free: **(800) 387 3405** Tel: **(905) 792 OTWO (6896)** Fax: **(905) 799 1339**

E-mail: [resuscitation@otwo.com](mailto:resuscitation@otwo.com) Website: [www.otwo.com](http://www.otwo.com)