

# O-TWO RESUSCITATION AND ANESTHESIA FACEMASKS

**o\_two** controlled  
ventilation™

	Ref N°	COMPONENT DEFINITION
<input type="checkbox"/>	<b>02FM5097</b>	Silicone Universal Mask
<input type="checkbox"/>	<b>02FM5098</b>	Silicone Medium Mask w/ Chin Cup # 2
<input type="checkbox"/>	<b>02FM5099</b>	Silicone Large Mask w/ Chin Cup # 3
<input type="checkbox"/>	<b>02FM5100</b>	Silicone Child Mask # 2
<input type="checkbox"/>	<b>02FM5101</b>	Silicone Infant Mask # 1
<input type="checkbox"/>	<b>02FM5102</b>	Silicone Neonate Mask # 0

## CAUTION

1. Federal law restricts this device to sale by or on the order of a physician.
2. Patient should be constantly monitored whenever this device is in use.
3. Store in clean dry conditions. Away from heat and light.

## DIRECTION FOR USE

1. Check for clear airflow passage prior to use.
2. Connect to breathing circuit or resuscitation device with appropriate fitting.
3. Make sure all connections are compatible and fit firmly.
4. Hold by hand or secure with head strap.

## CLEANING AND STERILIZATION

Sterilize the components using one of the following methods:

1. Autoclaving (134°C/273°F).
2. Ethylene oxide gas.
3. Commonly available sterilizing solutions which are used for equipment coming into contact with patients.

LOT



**O-TWO MEDICAL TECHNOLOGIES INC.**

45A Armthorpe Road, Brampton, Ontario, Canada L6T 5M4  
Tel. (905) 792-6896 Fax (905) 799-1339

[www.otwo.com](http://www.otwo.com)

Rev. 6 - Feb 2021