


o_two™ USER-FRIENDLY NEBULIZER

How to connect the Nebulizer to the *O-Two Single-Use CPAP*

To administer nebulized medication, insert the nebulizer 'T' piece between the facemask and CPAP valve, and attach calibrated oxygen flow tubing to the regulator's 9/16" DISS outlet. Adjust CPAP level using the therapy flow control at the oxygen therapy outlet. CPAP performance remains unaffected with the nebulizer inline and activated.

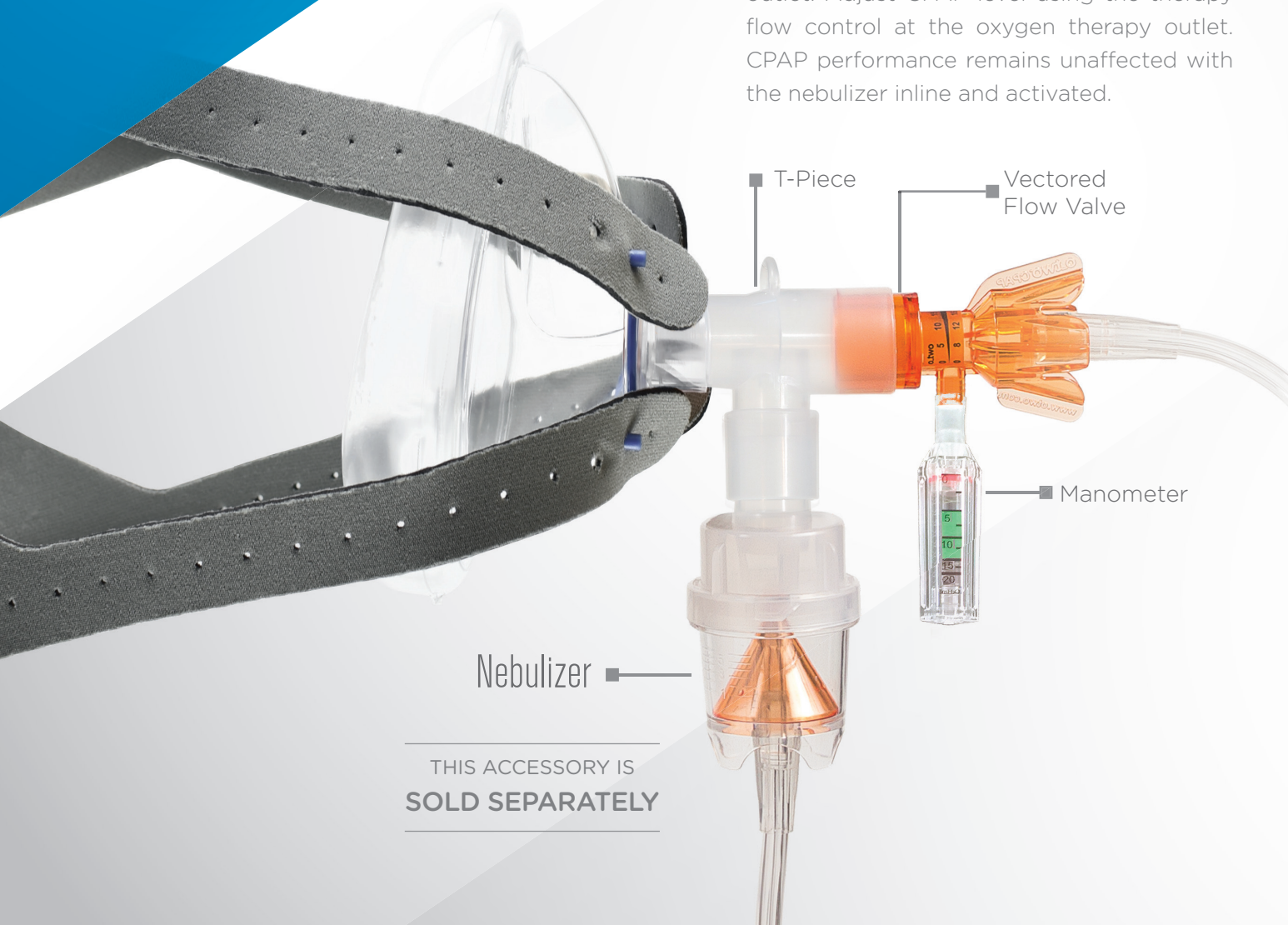

THIS ACCESSORY IS
SOLD SEPARATELY



ADVANTAGES

NO WASTED EQUIPMENT

Since the nabolizer is sold separately, and not always required, it can be retained for future use on other patients

LOW OXYGEN CONSUMPTION

Low flow CPAP combined with preset nebulizer flow reduces oxygen consumption compared to other devices

SIMPLE TO USE

Therapy flow control adjustment to set the CPAP level and "plug and play" for the nebulizer using the 9/16 DISS port on your regulator

PRESSURE MANOMETER

For verification of CPAP pressure

DECREASED WORK OF BREATHING

Open circuit imposes no work on the patient

COMFORTABLE AND SECURE HEAD HARNESS

Soft, conforming Neoprene material does not slip.

COST EFFECTIVE

Low purchase price and low oxygen consumption compared to other devices.