




O-TWO VENTILATION CIRCUITS

o_two controlled
ventilation

Ref N°	COMPONENT DEFINITION
<input type="checkbox"/> 01CV8010-CS	O-Two Disposable Ventilation Circuit (w/ PEEP port w/o anti-entrainment flapper) 
<input type="checkbox"/> 01CV8015-CS	O-Two Disposable Ventilation Circuit (w/ PEEP port & anti-entrainment flapper) 
<input type="checkbox"/> 01CV8016-CS	O-Two Disposable Ventilation Circuit (w/ PEEP port & anti-entrainment flapper) and 10' hose 

CAUTION

1. Federal law restricts this device to sale by or on the order of a physician.
2. Reuse of a single-use device may cause device malfunction and/or contamination.
3. Patient should be constantly monitored whenever this device is in use.
4. Store in clean dry conditions. Away from heat and light.

DIRECTION FOR USE

1. Check for clear air passage prior to use.
2. Connect to breathing circuit or resuscitation device with appropriate fitting.
3. Make sure all connections are compatible and fit firmly.
4. Hold by hand or secure with head strap.
5. Extend circuit to desired length before handling to patient.



CE 1639
15PL5506 Rev. 4
Feb-2021



O-TWO MEDICAL TECHNOLOGIES INC.

45A Armthorpe Road, Brampton, Ontario, Canada L6T 5M4
Tel. (905) 792-6896 Fax (905) 799-1339
www.otwo.com

EC **REP**

MedNet EC-REP GmbH
Borkstrasse, 10
48163 Münster, Germany