

RESCUE BREATHER[®] CPR POCKET VENTILATOR

RE-ORDER: #01PVI500W



Rev5 Dec 16, 2019



LOT



CONTENTS

1. Foldable seamless cushion mask with oxygen inlet
2. One way valve with filter
3. Elastic head strap


⚠ CAUTION ⚠

This product should be used by trained personnel.



O-TWO MEDICAL TECHNOLOGIES INC.

45A Armthorpe Road, Brampton
Ontario, Canada L6T 5M4
Tel. (905) 792-6896 Fax (905) 799-1339

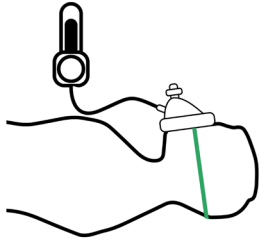


Connect the oxygen supply tubing when supplemental oxygen is required.


⚠ CAUTION ⚠

In the presence of oxygen, keep away from the presence of open flames, oil and grease


OPERATION FUNCTION

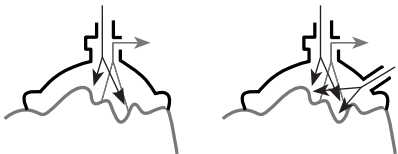


Inhalation/ Expiration resistance less than 3.0 cm H₂O at 50 lpm

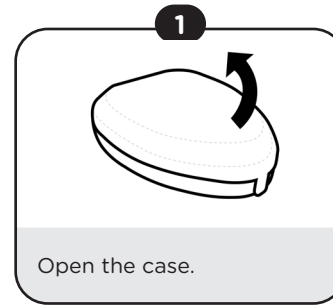

Open the case.

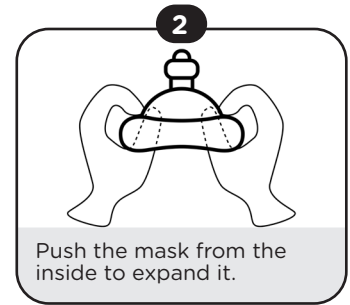

Push the mask from the inside to expand it.

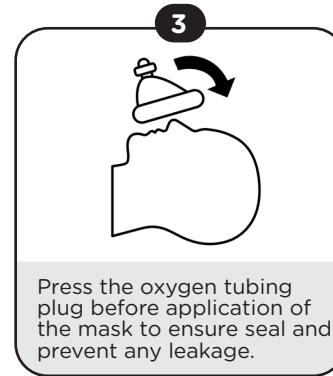


Press the oxygen tubing plug before application of the mask to ensure seal and prevent any leakage.



Give rescue breaths through the valve.



Continue CPR procedure accordingly.

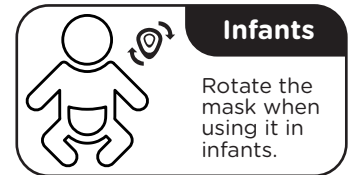

Infants

Rotate the mask when using it in infants.

www.otwo.com

o_two™