

RESCUE BREATHER

O1RB1000

o_two controlled ventilation

Mouth to Mouth RESUSCITATION AID (DISPOSABLE NOT FOR REUSE)

LOT 2015-11
PAT. N° 97/7008

EU Representative
Houben/Marcel
32 Rue Vin ve
4030 Li ge/Luik
Belgique



www.otwo.com

O-Two Medical Technologies Inc.
Brampton, Ontario - Canada
Tel: +1 905 792-6896
email: resuscitation@otwo.com

This resuscitator is designed to offer protection to both the first aider and the patient when performing Mouth to Mouth Resuscitation.

INSTRUCTIONS FOR USE

- 1 Ensure it is safe to approach the casualty.
- 2 If no breathing sounds or movement are detected, place casualty on their back.
- 3 Tilt head back and extend jaw forward.
See figure A
- 4 Clear airway of any obstruction and keep it clear.
- 5 Loosen clothing around the neck



- 6 Unfold the plastic sheet that surrounds the device and place the oval side of the mouth piece of the resuscitator in the casualty's mouth.
See figure B



- 7 With finger and thumb, pinch shut the nostrils through the sheet while supporting the jaw with the other hand.
See figure C



- 8 Make a good seal over the device and against the face. Commence mouth to mouth resuscitation as required. *See figure D.* Ensure that the mouth and airway are kept clear of vomit and or secretions.



WARNING Do not overinflate, take special care with children and babies



- 9 Continue the resuscitation until medical assistance arrives or until breathing recommences when casualty should be placed carefully in the recovery position.
- 10 Remove immediately after use and avoid any direct contact with the casualty's side of the device. Discard safely using original packaging.

NOTE In case of possible spinal/head injury, when opening the airway, use modified jaw thrust method without head tilt.

NOTE These instructions are a guide only and no substitute for proper training.