



e700

AUTOMATIC TRANSPORT
VENTILATOR

MULTIPLE MODES, COMPACT, LIGHTWEIGHT, EASY TO USE

The e700 ventilator ushers in a “new era” in controlled ventilation for resuscitation and patient transport!

These electronically controlled, pneumatically powered ventilators provide a range of ventilation solutions for all types of patient requirements and all levels of pre-hospital and in-hospital healthcare professionals.

LONG LASTING
BATTERY
18h - 24h

ECONOMICAL
Low gas consumption

LIGHTWEIGHT
2.4 kg / 5.29 lbs



SIMPLICITY

The ventilation solutions offered by the e700 cannot be compared to any other products of its type. The units are self-contained and only require attachment to a regulated oxygen supply and a transport ventilator circuit for immediate use. The easily replaceable, long lasting (18h - 24h) battery can be charged while inside the ventilator or can be removed for charging and quickly replaced by a fresh battery pack. The display lighting has adjustable brightness for easy visualization of the ventilator settings in any ambient light conditions. Designed for a range of patient sizes (from large adult to infant), the e700 comes in a very small and lightweight package.

SAFETY

The continuous monitoring of ventilation parameters ensures that the device is always fully functional and ready for immediate use. The wide range of both visual and audible alarms provides the healthcare professional with warnings of any changes in patient or device parameters. Correction of any alarm is simple to achieve due to the compact and easy to operate control layout. The Intuitive Patient Apnea Backup (IPAB) mode provides additional security for the spontaneously breathing patient on CPAP by automatically commencing ventilations should their inspiratory efforts cease.

FUNCTIONALITY

The simplicity of operation masks the level of sophistication offered by the e700, providing ventilation for both resuscitation and transport with the minimum of control adjustments required for simple patient set up. The range of ventilation modes offered provides “ICU level” patient care while maintaining the ease of operation for all patient care and transport situations in the pre-hospital and in-hospital setting. The inclusion of ventilation modes for “mask or protected airway CPR”, with visual and audible prompts, adds another dimension to the application for these products not found on other ventilators.

CONTROLLABILITY

The operation of the controls of the e700 provide an ease of use concept that is second to none. These products are designed to speed up and simplify even the most complex ventilation set up. There are no multiple screens to scroll through to establish patient ventilation parameters. The units have an initial, pre-set, start-up mode requiring only the selection of patient size to begin ventilation.

ECONOMY

In addition to the patient care benefits, the e700 ventilator provides excellent low gas consumption and an extremely long battery operating time. This assists in providing controlled ventilation to the patient over an extended period, making the e700 ventilator ideal for long transports where both electrical and oxygen supplies are always a critical concern.

SPECIFICATIONS

| | | |
|---|-----------------------------------|--|
| DEVICE CLASS PER MDD | | II b |
| CLASSIFICATION PER IEC60601-1 | Protection against electric shock | Class II |
| | Protection against electric shock | Type BF |
| | Protection against water | IP X4 |
| POWER SOURCE | | Compressed Oxygen, 45 to 87 PSI (3-6 Bar) |
| CIRCUIT CONTROL SOURCE | | Electric |
| VENTILATION MODES | | A/C (VCV,PCV), SIMV w/ PSV, BiLVL w/ PSV, CPAP w/ PSV, Mask CPR and Intubated CPR |
| SUPPORTING VENTILATION | | PSV: 0, 4-35 cmH ₂ O (± 10% or ± 2 cmH ₂ O) |
| VENTILATION RATE | | 5 - 60 (± 10% or ± 1BPM) |
| MINUTE VOLUME (L) | | Calculated |
| TIDAL VOLUME (ML) | | 50 - 2000 ± (4ml + 15%) BTPS * |
| TIDAL VOLUME IN CPR MODE (ML) | | 50 - 1400 ± (4ml + 15%) BTPS * |
| MAXIMUM DELIVERED FLOW (L/MIN) | | 100 - 120 |
| MANUALLY TRIGGERED VENTILATION | | Yes, set flow rate or pressure will be delivered during I time then Inspiratory hold |
| MAXIMUM INSPIRATORY HOLD TIME | | 6 sec. |
| I:E RATIO | | 1:4 - 3:1 (± 20%) |
| PEEP (CM H2O) | | 0,4-20 (± 10% or ± 2 cmH ₂ O) |
| PSV | | OFF, 4 - 35 (± 10% or ± 2 cmH ₂ O) |
| CPAP (CM H2O) | | 4-20 (± 10% or ± 2 cmH ₂ O) |
| O2 (%) | | 60 or 100 (± 15%) |
| PMAX (CM H2O) | | 10 - 80 (± 10% or ± 2 cmH ₂ O) |
| PMIN (CM H2O) | | 0 - 20 (± 10% or ± 2 cmH ₂ O) |
| PCV (CM H2O) | | 4-50 (± 10% or ± 2 cmH ₂ O) |
| TI (SEC.) | | 0.2 - 9 (± 20%) |
| TRIGGER SENSITIVITY (L/MIN) | | 1 -15, or 2 cmH ₂ O below baseline in CPAP mode only |
| INHALATION PRESSURE (CM H2O) | | 4-50 (± 10% or ± 2 cmH ₂ O) |
| PRESSURE VENTILATION TERMINATION | | 20% - 80% of max. Flow |
| APNEA BACK UP TIME (SEC.) | | 10-60 (± 0.5s) |
| BATTERY OPERATING TIME AT ROOM TEMPERATURE (HRS.) | | > 18 hrs for default settings (Data obtained using fully charged new battery) |
| ALTITUDE COMPENSATION | | up to 4000m (13000 feet) |
| BATTERY HOT SWAP | | No |
| BUILT-IN BATTERY CHARGER | | Yes |
| AC/DC POWER SUPPLY | | 100-240 VAC/ 19 VDC, 4.74 A |
| PATIENT CIRCUIT | | O-Two Electronic Ventilator Circuit |
| MOUNTING BRACKET | | Mounting brackets for road ambulance and mobile setting |
| DISPLAY | | 4.3" Color TFT |
| LIVE MONITORING | | Mve,Vte,Paw(AV),PAW(Peak), Rate (bpm), Battery level |
| REAL TIME WAVEFORM | | Pressure or Flow |
| DAY/NIGHT DISPLAY MODE | | Yes |
| PARAMETER SETTINGS | | Control Selection Knob |
| LOCK KEY FUNCTION | | Yes |
| PAUSE FUNCTION | | Yes |
| NOISE LEVEL IN NORMAL USE | | Less than 65 dBA |

| | | | |
|--|-----------------|-----------|---|
| ALARMS (VISUAL AND AUDIBLE) | | | Gas Supply Pressure, Airway Pressure limits, Minute Volume limits, Battery status, APNEA, Breathing Circuit Integrity, Leakage and Blockage |
| AUDIBLE SILENCE | | | Yes, 120 second max |
| DIMENSIONS (MM) | | | 250 x 200 x 155 |
| WEIGHT (KG/LBS) WITH/WITHOUT BATTERY | | | 2.4/1.77 5.29/3.9 |
| INTERNAL VOLUME OF THE COMPLETE RESPIRATORY SYSTEM (REUSABLE AND DISPOSABLE) | | | approx. 690 ml without mask approx. 800 ml with mask |
| DEAD SPACE OF PATIENT VALVE WITH ELBOW | | | Approx. 35 ml |
| COMPLIANCE (DISPOSABLE) HOSE SYSTEM | | | 16.6 ml/kPa |
| RESISTANCE OF PATIENT HOSE SYSTEM (INHALATION AND EXHALATION): | | | Less than 6 cmH ₂ O at 60 l/min & Less than 6 cmH ₂ O at 30 l/min |
| ENVIRONMENT CONDITION | Ventilator | Operating | - 18°C to +50°C, Rh: 15% to 95% |
| | | Storage | - 40°C to +60°C, Rh: 15% to 95% |
| | Battery Pack | Charge | 0°C to +40°C |
| | | Discharge | - 20°C to +60°C |
| | Patient Circuit | Storage | - 20°C to +35°C, low humidity and no corrosive gas atmosphere. |
| | | Operating | - 18°C to +50°C, Rh: 15% to 95% |
| | Patient Circuit | Storage | - 20°C to +60°C, Rh: 15% to 95% |
| | | Operating | - 18°C to +50°C, Rh: 15% to 95% |

* BTPS: Volume measurements corrected to Body temperature 37°C and Barometric pressure 101.3Kpa under saturated conditions (100% Humidity). Note: Measurement uncertainty: 5% for volume parameters and 6% for pressure parameters.

ORDERING INFORMATION

| | | |
|---------------|--|---------|
| 01EVE700 | e700 - Electronic Automatic Transport Ventilator c/w Disposable Patient Circuit, Resuscitation Mask, 6 Foot Supply Hose*, Calibrated Test Lung and Power Supply (Specify Country of Use) | Each |
| 01CV8030-CS | O-Two Medical Single-Use Electronic Transport Ventilator 6 Foot Circuit with Protective Sleeve. For e500, e600 and e700 | Case/10 |
| 01CV7035 | O-Two Medical "e" Series Ventilator "Smart Mount" Multi-configuration Mounting Bracket (Ambulance Cot, Hospital Stretcher, Bed, Roll Stand) | Each |
| 01CV8040-CS | O-Two Medical "e" Series Ventilator Replacement Intake Filter/Cover | Case/10 |
| 01TA1852 | O-Two Medical "e" Series Ventilator Replacement 1 Litre Test Lung with Compliance Restrictor | Each |
| 01FV4303-DISS | O-Two 6 Foot (1.85 Meter) O2 Supply Hose with 9/16 DISS Nut and 9/16" DISS Nut Ventilator Connection | Each |
| 01CV9100 | O-Two Medical "e" Series Replacement Lithium Ion Replacement Battery | Each |
| 01CV0105 | Power Supply - eSeries | Each |
| 01CV0106 | Power supply cord for eSeries power supply | Each |
| 01CV0102-EU | Power supply cord for eSeries power supply | Each |
| 01CV7050 | eSeries Automatic Transport Ventilator Carrying Case - With sling-style shoulder strap (specifically designed for eSeries) | Each |
| 01TA7650 | Leak test kit for e-vents | Each |

THIS PRODUCT HAS A TWO YEAR WARRANTY AGAINST MANUFACTURERS DEFECTS.